

Teladoc Health™ Vita™ Quick Reference Guide

Operating the Vita



Vita User Guide

Scan the QR Code with your smart phone to access the Vita User Guide.

Charging Vita

The Vita battery lasts 4-5 hours on a full charge (depending on usage).


- Vita will charge from 0% to 80% in two hours.
- Vita will charge from 0% to 100% in six hours.

NOTE: If the Dock is not available for any reason and Vita needs to be charged, plug the Vita into a hospital grade grounded 115VAC 10Amp, or 230VAC 5A outlet.

Power On Sequence



1. Switch the Rocker Switch **0 I** to the on position.
2. Plug Vita in at its docking station or plug the power cord into an approved wall socket.
3. Turn on the power to the system by pressing the computer power Button **⏻** for approximately 1 second.

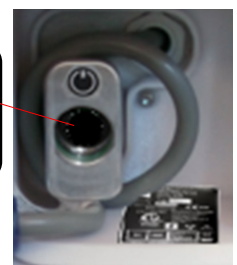NOTE: Computer Power Button is located under the door on back of the Vita.




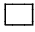





Teladoc Health™ Vita™ Quick Reference Guide

Power Off Sequence






1. Press power button  and wait 30-45 seconds for screens and LED indicators on the base to turn off.
2. Turn off the Rocker Switch. 


CAUTION: Verify the LED indicators on the base are off before turning off the Rocker Switch.

Vita Status Indicator Lights



Vita Status	LED Color, Animation	
Start Up	Gray, static	
Idle (not charging)	White, static	
Idle (charging)	White, sinusoidal	
Message Mode	Magenta, static	
Mission—AutoDRIVE command from the Chest Display, destination other than Dock.	Blue, static	
Session	Teal, static	
Fault Condition	Yellow, static or flashing	


Chest Display Out of Session Functions

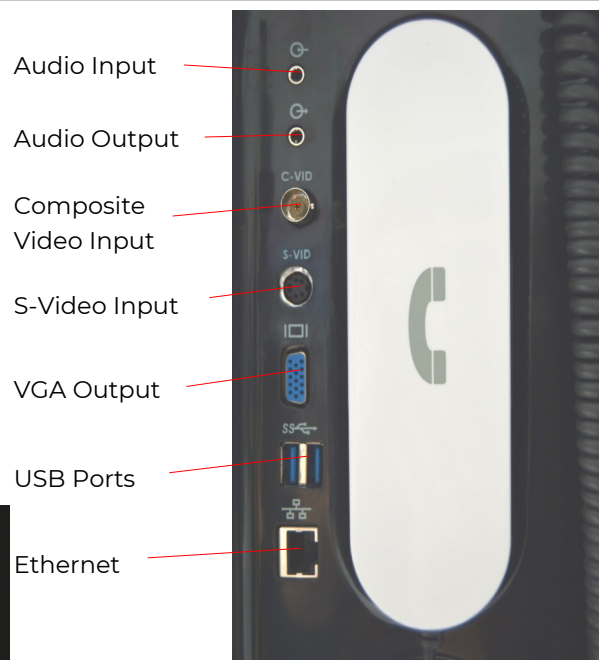Icon	Function
	Exit —Puts Vita into Idle state, with animation on the screens.
	AutoDRIVE —Choose a pre-defined destination for the Vita to travel to.
	Camera —Tap to preview any of the cameras connected to the device.
	Settings — Access device settings
	Info —Tap to display information about the Vita



Vita Cleaning Mode


Tap Settings  > Cleaning Mode  to disable the screen for 30 seconds to allow cleaning.

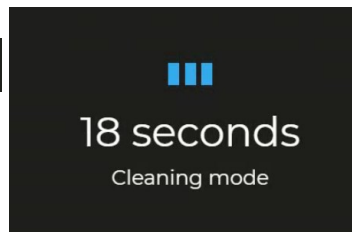

See the Vita User Guide for approved cleaners.

Device Network Checkup

The Network Checkup screen runs and displays the results of a test of the device's current network connection.

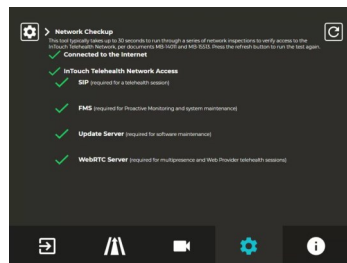

Tap Settings  > Network Checkup  to run the Network Checkup.


Emergency Stop Button

To disable Vita movement:

- Press the red Emergency Stop Button.
- The Vita can now be pushed as needed.

To enable Vita movement again:

- Twist the red Emergency Stop Button clockwise 1/4 turn until it pops out.

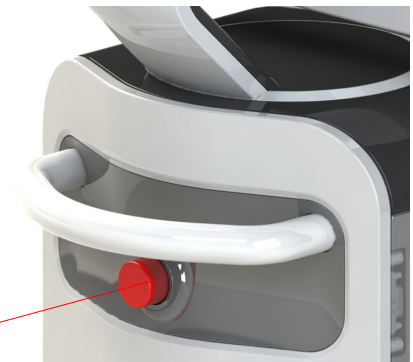

Emergency Stop