

Teladoc Health™ Lite® V3 Quick Reference Guide


Operating the Lite V3



To turn on the Lite, press **POWER** once. The display will power up in 1-2 minutes.

To turn off the Lite, press **POWER** and follow on-screen directions.

MUTE Button
Mutes audio from the Lite.

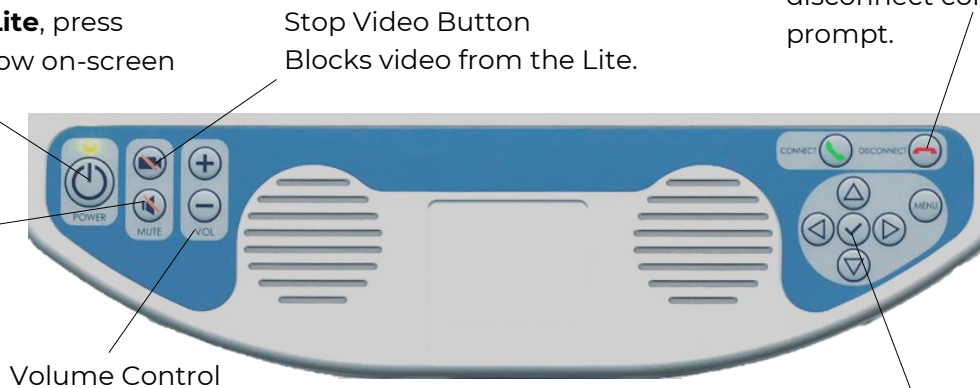

Press **DISCONNECT** to end an active session following a disconnect confirmation prompt.

Moving the Lite

The Lite V3 is designed to be moved by hospital staff into a broad array of locations ranging from clinics, emergency rooms, rural hospitals, long-term care communities and more.

- Ensure Lite is set at its lowest position.
- Ensure any auxiliary devices attached to Lite are unplugged.
- Ensure the power cord is unplugged.
- Ensure the wheels are unlocked before moving.
- Exercise caution when encountering thresholds.
- Move it to the desired location.
- Lock the wheels.
- Plug in the power cord.

The Lite V3 requires a hospital grade grounded AC outlet supplying a minimum of 100-240 VAC, 50/60 Hz, 6.5A.

NOTE: A battery charge can last 7 hours.

Menu & Navigation Buttons

Pressing the Menu button will enable out of session interaction with the Lite. Use the arrow keys to navigate to the desired function, and press the Check button to select.



Adjusting the Height

The Lite Head can be placed between 54.5 inches and 76 inches off the floor.

Lift or raise the head to adjust the height.

Lift or Lower Head Display



Teladoc Health™ Lite® V3 Quick Reference Guide

Power On Sequence

1. Make sure the rocker power switch, located under the base of the Lite V3, is in the on position.
2. Plug in the power cord and wait 20 seconds before powering on the Lite V3.

NOTE: Ensure that the power cord is plugged into a grounded hospital grade wall socket.

3. Press the power button once.

After 1-2 minutes of self-testing, a screensaver will appear on the Lite V3 display.




Power Off Sequence

CAUTION: Only power off when instructed by Teladoc Health Technical Service.

1. Press the power button once.
2. Press the check button within seven seconds after the “Do Not Turn Off” message appears.

The Lite V3 powers off after approximately 30-45 seconds.

Lite V3 On Screen Navigation


Icon	Function
	Tap to preview any of the cameras connected to the device.
	Tap to open device settings.
	Tap to display information about the Lite V3.



Lite V3 User Guide



Scan the QR Code with your smart phone to access the Lite V3 User Guide.

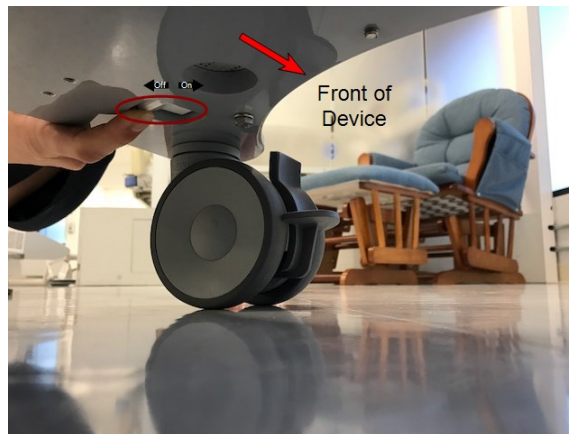NOTE: The Lite should be plugged in and left powered on at all times.



Device Network Checkup

The Network Checkup screen runs and displays the results of a test of the device's current network connection.

Tap Settings  > Network Checkup  to run the Network Checkup.

