


# Teladoc Health™ Mini™ Cart Quick Reference Guide

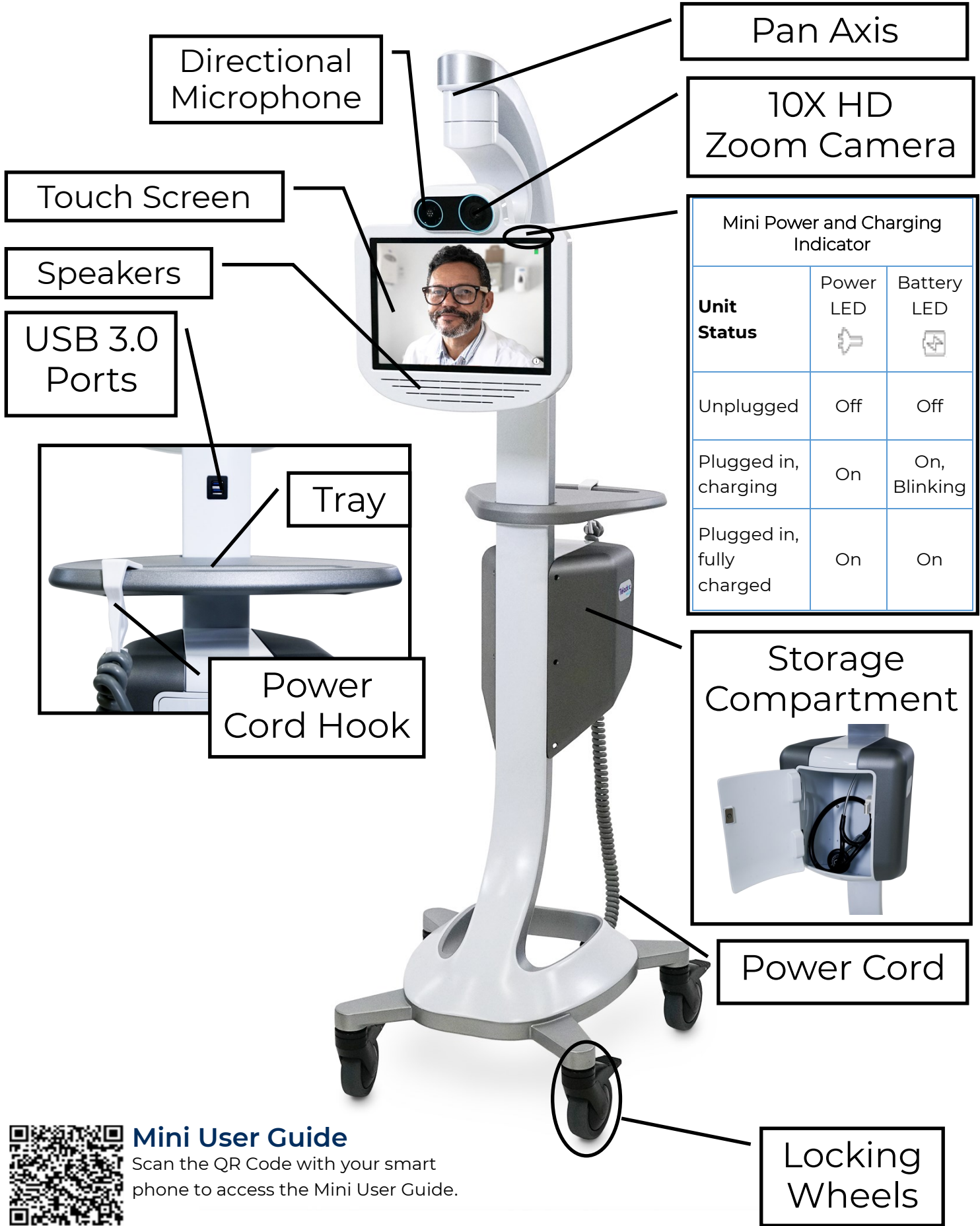

Pan Axis

10X HD Zoom Camera

Directional Microphone


Mini Power and Charging Indicator

Unit Status	Power LED	Battery LED
Unplugged	Off	Off
Plugged in, charging	On	On, Blinking
Plugged in, fully charged	On	On

Touch Screen

Speakers

USB 3.0 Ports

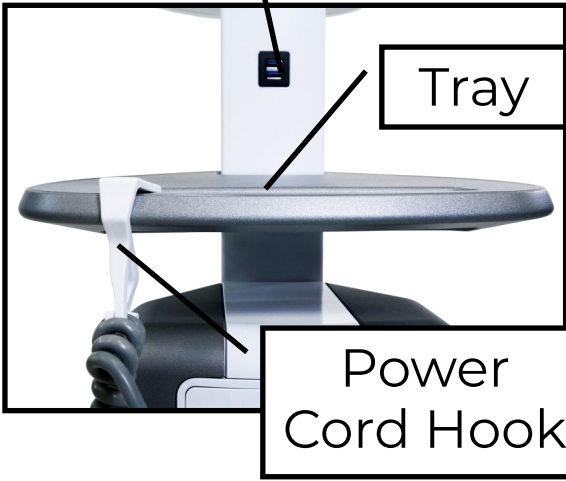

Tray

Power Cord Hook



Storage Compartment

Power Cord

Locking Wheels



**Mini User Guide**  
Scan the QR Code with your smart phone to access the Mini User Guide.

# Teladoc Health™ Mini™ Quick Reference Guide

## Powering On the Mini





Press and hold the power button for 2-3 seconds until the device powers on.


## Powering Off the Mini

1. Press and hold the power button for 2-3 seconds.
2. Slide the power-off slider on the screen when prompted.



**NOTE:** It is recommended that the Mini remain powered on to receive updates and to be available for use.

## Mini On Screen Navigation

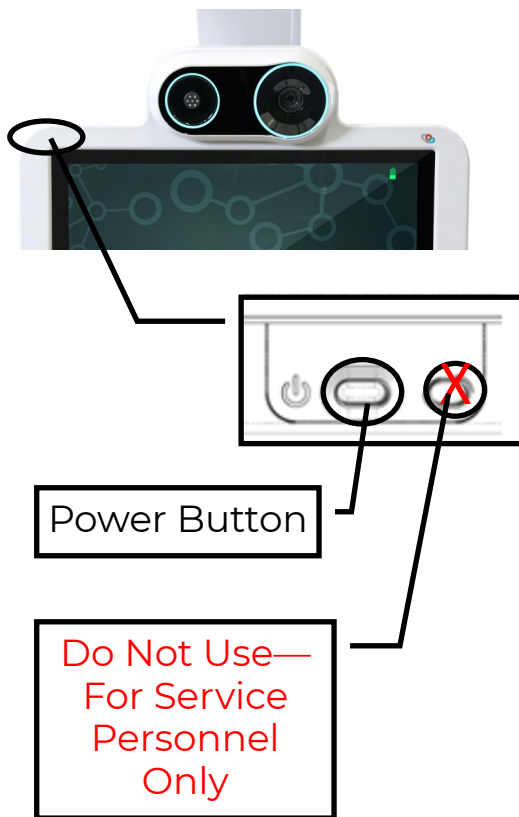
Icon	Function
	Tap to preview any of the cameras connected to the device.
	Tap to open device settings.
	Tap to open the device's User Guide.
	Tap to display information about the Mini



## Mini Cleaning Mode



Tap Settings  > Cleaning Mode,  to disable the screen for 30-seconds to allow cleaning.

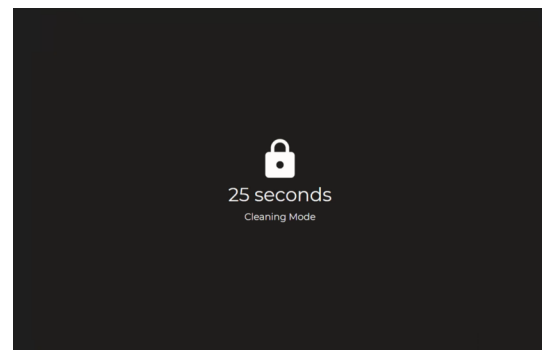See the Mini User Guide for approved cleaners.



## Device Network Checkup

The Network Checkup screen runs and displays the results of a test of the device's current network connection.

Tap Settings  > More Options  > Network Checkup  to run the Network Checkup.

