


Teladoc Health™ Vici® Quick Reference Guide

Vici At A Glance

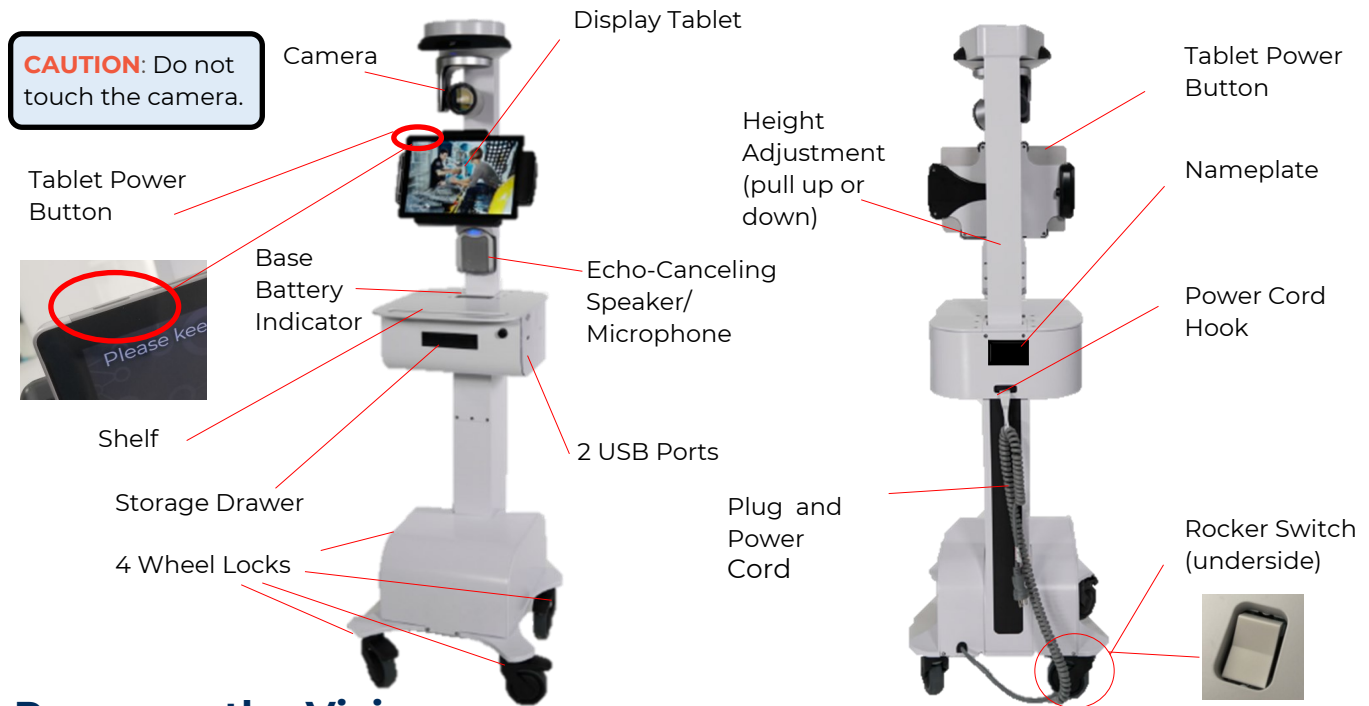

Power on the Vici

NOTE: Refer to MA-20053 *Vici User Guide* if the Vici is not powered on.

Press the tablet power button to power on the Vici tablet.

- The software is pre-installed and will launch on startup.
- The screen to the right displays when tablet is on and the software initialized.

CAUTION: Keep the Vici device **plugged in** whenever possible to keep it fully charged and ready for use.




Vici On Screen Navigation

Icon	Function
	Tap to preview any of the cameras connected to the device.
	Tap to open device settings.
	Tap to open the device's User Guide.
	Tap to display information about the Vici.

Vici Cleaning Mode

Tap Settings > Cleaning Mode to disable the screen for 30-seconds to allow for cleaning.

NOTE: See the Vici User Guide for approved cleaners.

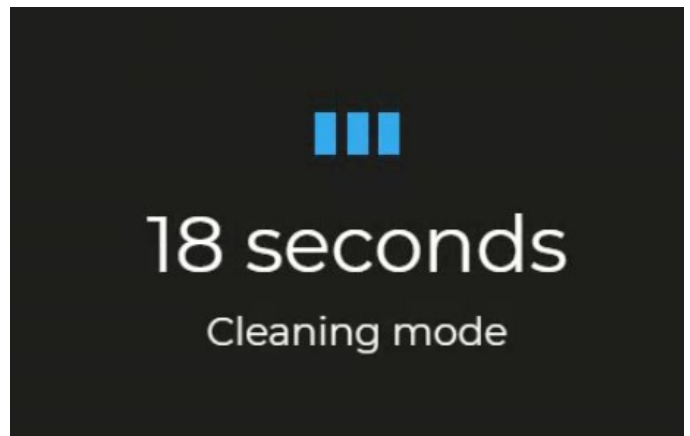

Vici® Quick Reference Guide

Adjust the tablet for easier viewing by tilting the monitor backward or forward.



CAUTION: Do not touch the camera.

Adjust the height by gripping the column and manually adjusting the column up or down (the position is held at any point on the column).

CAUTION: Vici should be powered on and charging whenever it is not in use.

Moving the Vici

Before moving:

1. Set the Vici at its lowest height,
2. Unplug any attached auxiliary devices.
3. Unplug the power cord and secure it on the power cord hook.
4. Unlock the wheels before moving.
5. Move the Vici to the desired location.
6. Lock the wheels.
7. Plug in the power cord.

WARNINGS:

- Wheels may lock when Vici is in motion if lock is engaged accidentally.
- Vici may tip forward if pushed with too much force when wheels lock.



Adjusting Speaker Volume and Muting Microphone

Adjust the Vici speaker volume by pressing the **up** or **down** volume buttons on the Integrated Speaker Microphone.

Mute the microphone by pressing the **center** button.

- The lights on the device flash when microphone is muted.

Device Network Checkup

The Network Checkup screen runs and displays the results of a test of the device's current network connection.

Tap Settings > Network Checkup to run the Network Checkup.

